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“磊仕” 逸活鼻面罩
“Respironics” EasyLife Nasal Mask

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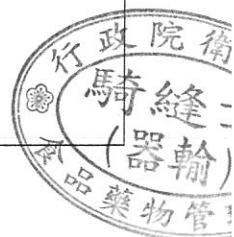
Treatment	Exposure Time	Active Ingredients	Solution	Rinse	Maximum Cycles
Cidex Activated Dialdehyde Solution	45 minutes	2.4% Glutaraldehyde	Full Strength	Potable Water, 5 Liters per Mask(minimum)	30
or					
Cidex OPA	12 minutes	0.55% Ortho-Phthalaldehyde	Full Strength	Potable Water, 5 Liters per Mask(minimum)	30
or					
Wavicide	45 minutes	2.65% Glutaraldehyde	Full Strength	Potable Water, 5 Liters per Mask(minimum)	30
or					
Control III	10 minutes	10% Alkyl Dimethyl Benzyl Ammonium Chloride 10% Alkyl Dimethyl Ethyl Benzyl Ammonium Chloride	8 ml per liter	Potable Water, 5 Liters per Mask(minimum)	30

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Treatment	Exposure Time	Maximum Cycles
Sterrad 100S	Machine Controlled	30

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EasyLife Mask	Petite	Small	Medium	Medium Wide	Large
Mask with headgear	1050000	1050001	1050002	1050003	1050004
Mask only	1050010	1050011	1050012	1050013	1050014
DuoPack: includes headgear and two cushion sets of the same size	1050005	1050006	1050007	1050008	1050009
FitPack: includes headgear and cushion sets* in two sizes	N/A	1050078 small and medium		1050079 medium wide and large	

Replacement parts	Part number
Petite cushion set	1050090
Small cushion set	1050091
Medium cushion set	1050092
Medium wide cushion set	1050093
Large cushion set	1050094

Replacement parts	Part number
EasyLife headgear	1050087
EasyLife headgear, reduced size	1050086
Headgear clips (pack of 2)	1050097

cushion set 包含 1 個 inner seal cushion 及 1 個 outer support cushion

製造廠名稱/地址:Manufactured by Respironics Medical Products (Shenzhen) Co., Ltd. (地址：Blocks 6&7, Tong Xi Chong 2nd Industrial District, Tong Xi Chong Village, Bao An District, Shenzhen, China 518105) for Respironics, Inc. (地址：1001 Murry Ridge Lane Murrysville, PA 15668, U.S.A.)
藥商名稱：台灣飛利浦股份有限公司
藥商地址：台北市南港區園區街 3-1 號 15 樓